

SAM® Pelvic Sling II

SLIDE. CLICK. SECURE.

OPTIMAL FORCE STABILIZATION

SAM* Pelvic Sling is the first and only force-controlled circumferential pelvic belt. Featuring an optimized one-piece design, it's scientifically proven to safely and effectively reduce and stabilize open-book pelvic ring fractures.









FEATURED TECHNOLOGY

AUTOSTOP BUCKLE

The patented SAM® AUTOSTOP buckle acts to avoid over- or undertightening of SAM® Pelvic Sling. Once optimal compressive force is reached, two prongs activate, clicking to confirm correct application. The sling is secured in place with its hook and loop system strap.





SAM[®] Pelvic Sling II

ENGINEERED TO PRESERVE LIFE

LOW-FRICTION POSTERIOR SLIDER

Easier transfers following severe trauma made possible by lowfriction materials covering backside of the pelvic sling.

2 SAM® AUTOSTOP BUCKLE

Tactile and audible "click" provides clear feedback, confirming proper application and preventing over-tightening while maintaining optimal force.

PRECISION STRAP

Precisely-engineered holes lock with AUTOSTOP Buckle, allowing for quick application. Secure fastening with sturdy hook and loop strap.

4 TAPERED DESIGN


Narrow and tapered front facilitates urinary catheterization, interventional radiology, external fixation, and abdominal surgery

5 RADIOLUCENT MATERIAL

All materials are radiolucent except the buckle's two stainless steel springs, allowing X-rays and CT-scans without removal.

6 3 SIZES AVAILABLE

The Standard size fits 95% of the population. Complete the set with the Small and Large sizes to fit almost any body type.

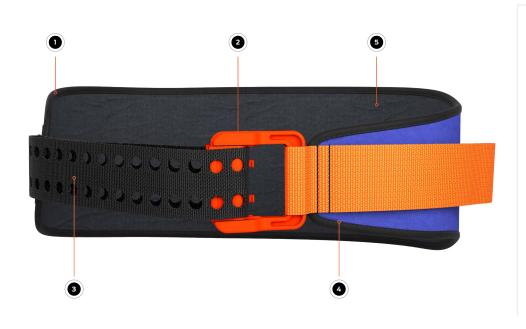

MODELS







OD GREEN /BLACK

GLOBAL SALES

U.S. Toll Free: 800.818.4726

U.S. Local: 503.639.5474

U.S. Fax: 503.639.5425

sammedical.com

PRODUCT SPECIFICATIONS

PACKAGED SIZE & WEIGHT

Height: 5.5 in (14 cm) Width: 7.5 in (19 cm) Depth: 3.75 in (9.5 cm) Weight: 8.6 - 9.2 oz (244 - 250g)**

EC REP

OPERATING PRINCIPLE

Optimal force is reached when 150N of circumferential pressure is attained, which extends prongs from SAM® AUTOSTOP Buckle through the Precision Strap.





Emergo Europe, Prinsessegracht 20, 2514 AP, The Hague, The Netherlands SAM Medical Products⁶ 27350 SW 95th Ave. Ste 3038 Wilsonville, OR 97070 USA

MODELS AVAILABLE

Civilian

Small 27-45"/69-114 cm (SL556652-SM) Standard 32-50"/81-127 cm (SL556652) Large 36-54"/91-137 cm (SL556652-LG)

Military Standard 32-50"/81-127 cm (SL556652-OD)

* Size and weight vary by configuration and manufacturing process.

**Weight varies by size (Small, Standard, Large)
SAM® is a registered trademark in the United States and other countries. ©2017 SAM Medical Products. All rights reserved.