

SAFETY LANCETS

Safe: No possibility to reuse Easy to handle: Intuitive 2 step operation Gentle: Electro-polished and silicone-coated needle is linearly driven to assure lower pain and quick healing Sterile: Needles are gammasterilized, fully covered and protected with a sterility tab Reliable: Risk of misuse or receiving an incorrect blood sample is eliminated through a unique, patented, self-destructive activation mechanism Efficient: an adequate blood sample with just a single push of the button




KEY FEATURES OF ACTI-LANCE

CE 0344

- *Easy to handle: intuitive activation, low firing force, no lancet loading required, 2 step operation only*
- Minimized pain: ultra sharp and silicone-coated needle, double spring mechanism to ensure high puncture speed and eliminate needle vibrations
- Sufficient to lance once: different color-coded versions, right needle exposure length, adequate blood flow
- Self-destructive mechanism excluding possibility of reuse, sterilized reliable sterility protector



INSTRUCTIONS FOR USE

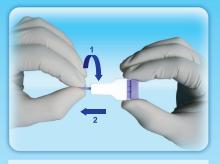

Twist off the protective cap and pull it straight out.



Place Acti-Lance[®] on the desired puncture site and press the top push button to activate the device



Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

TECHNICAL DATA & ORDERING INFORMATION



Acti-Lance[®] Lite is ideal for blood glucose monitoring purposes and is especially recommended for all patients with delicate skin including children.



Acti-Lance[®] Universal is perfect for most of the common tests done with capillary blood including blood glucose, cholesterol, hemoglobin, blood group classification, coagulation tests, virology tests and many others.



Acti-Lance[®] Special assures larger blood samples for the widest variety of clinical purposes. It is also perfect for heel sticks on neonates as well as finger sticks on adults in cases where the test requires more blood.



 Manufactured by:

 HTL-STREFA, S.A.

 Adamówek 7,

 95-035 Ozorków, Poland

 T: +48 22 587 05 71, F: +48 22 811 05 36

 info@htl-strefa.pl, www.htl-strefa.pl

Distributed in the US by: HTL-STREFA, Inc.

3005 Chastain Meadows Pkwy, Suite 300, Marietta, GA 30066, USA T: +1 770 528 0410, F: +1 770 528 0411 info@htl-strefa.com, www.htl-strefa.com for more information go to: www.acti-lance.com