

Positioning line for correct placement

Including knee fixation

3 sizes available




Pelvic Sling

Pneumatic system for the stabilisation of pelvic fractures

/ Pelvic Sling

The Pelvic Sling is a non-invasive, external pelvis stabiliser for compression and stabilisation of unstable pelvic fractures. The inflatable cuffs exert a lateral pressure to the pelvis for external stabilisation of the pelvic ring to restore the anatomic conditions. Hemorrhage is reduced by reduction of the intra-pelvic volume.

Features

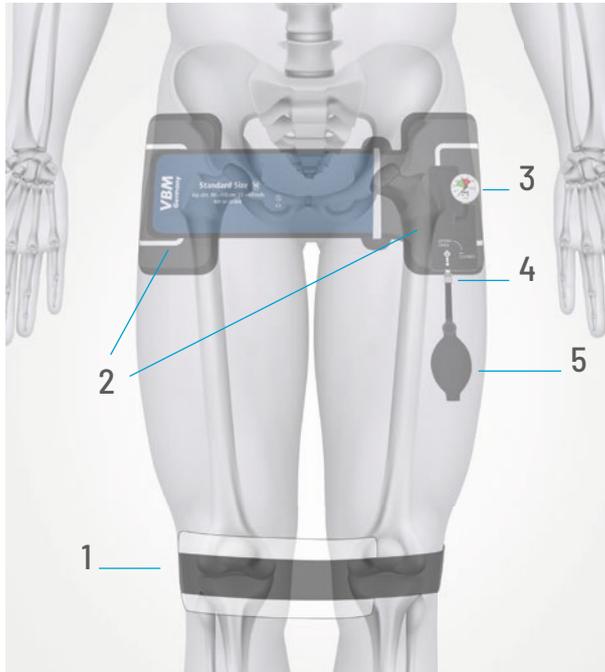

Pelvic Sling in situ with slide splint used as knee fixation.

1 Slide Splint / Knee Fixation

- To slide the Pelvic Sling underneath the patients pelvis
- After removal used as knee fixation

2 Cuffs

- Lateral compression of the pelvis due to pneumatic cuffs
- Minimal mechanical manipulation on the fractured pelvis

3 Manometer

- Visual control of the applied pressure

4 Stopcock

- For a leak free transport

5 Hand Inflator

- For inflation and deflation of both cuffs
- Removable for MRI use



Positioning

- Due to the positioning line at the level of the trochanter major a correct placement of the Pelvic Sling can be achieved
- The slide splint facilitates the application of the Pelvic Sling and can help to consider the important internal rotation of the lower extremities

Order information

Pelvic Sling / For single use

Size	Patient	Hip Circumference	REF	Box
S	Junior	70 – 90 cm	34-10-006	1
M	Standard	90 – 110 cm	34-10-004	1
L	Large	110 – 140 cm	34-10-005	1

The Pelvic Sling is packed in a compact size bag, ready to use.

Pack size:

42 cm (L) x 19 cm (W) x 7 cm (H)



www.penncare.net
sales@penncare.net
 1-800-392-7233